

MYSOZO & SOZOAPP CLINICIAN


Delete Measurement

1

Select Measurement Date

1. Sign into MySOZO or SOZOapp as a clinician
2. Find and select the patient whose measurement you wish to delete
3. From the measurement list, select the measurement you want to delete


! "Measure" is only available through tablet

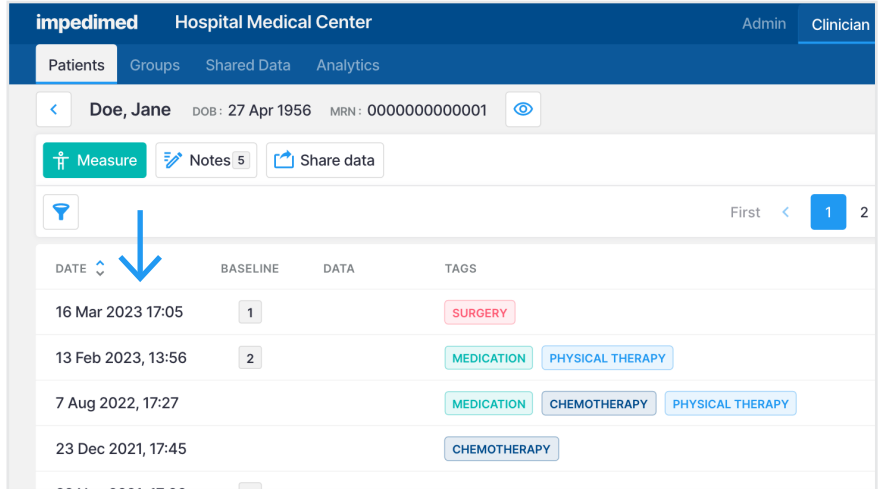

2

Delete Measurement

1. Review the results and confirm that this is the measurement you want to delete
2. Select the "Delete" button

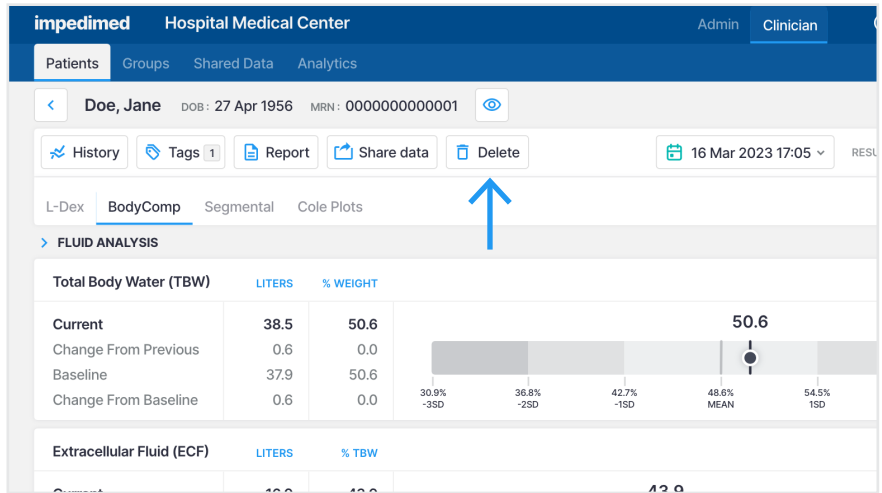

3

Confirm Deletion

1. Select the checkbox on the confirmation pop-up
2. Select "Delete"

By selecting delete option a success notification appears.

Please note: Measurement cannot be restored. If measurement is currently used as baseline, the impacted baseline will be cleared.

