


Setting the Baseline for L-Dex®

1

Select Measurement Date

1. Sign into MySOZO or SOZOapp as a clinician
2. Find and select a patient
3. From the measurement list, select the measurement to be used as a baseline


! "Measure" is only available through tablet

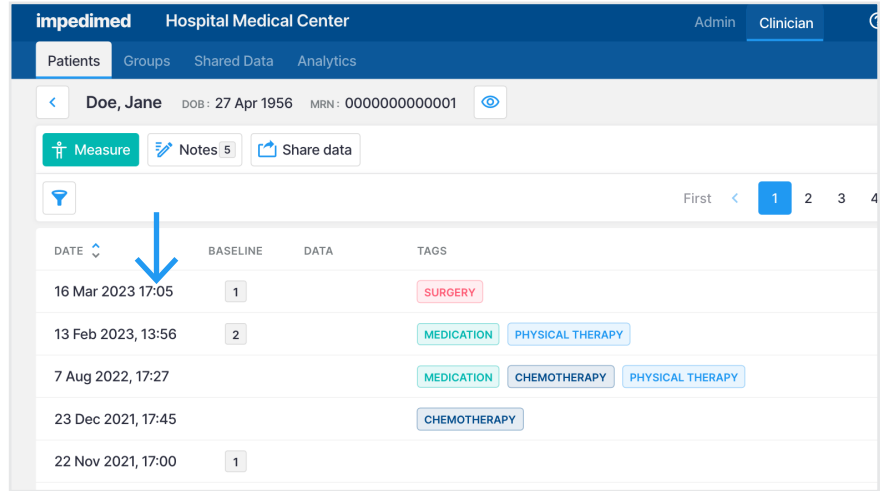

2

Select L-Dex

1. From the results screen, select the L-Dex tab

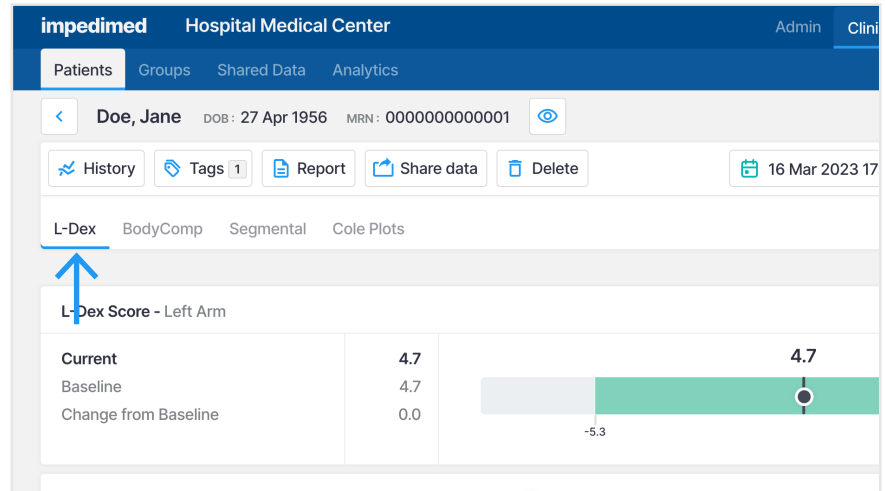


3

Set Baseline

1. Select the "Set As Baseline" check box

By setting the baseline, a success notification appears. To unset the baseline deselect the check box.

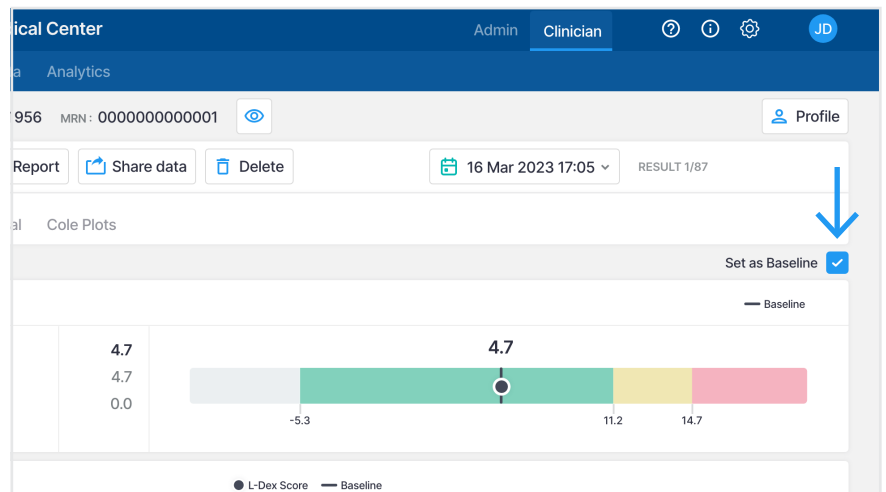


Setting the Baseline for BodyComp™

1

Select Measurement Date

1. Sign into MySOZO or SOZOapp as a clinician
2. Find and select a patient
3. From the measurement list, select the measurement to be used as a baseline


! "Measure" is only available through tablet

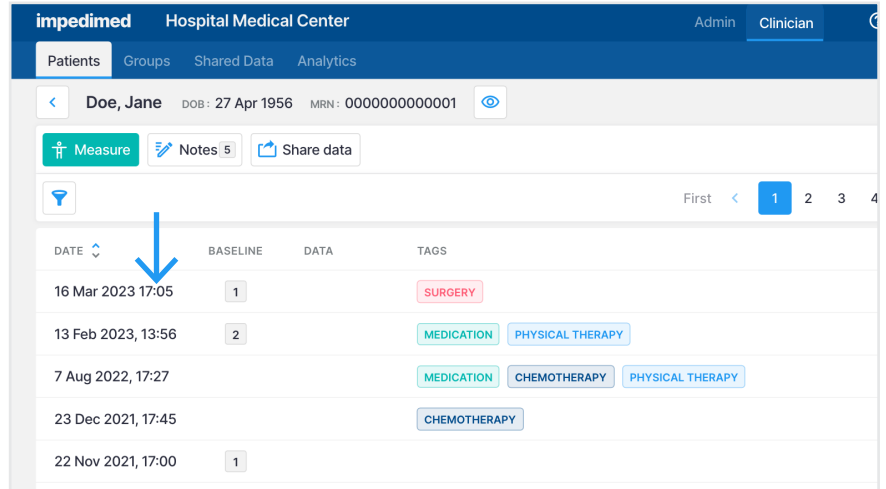

2

Select BodyComp

1. From the results screen, select the BodyComp tab

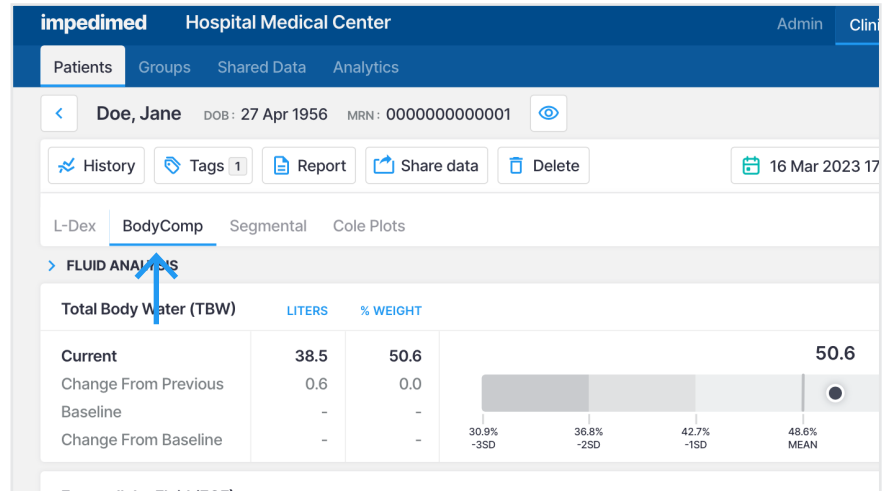


3

Set Baseline

1. Select the "Set As Baseline" check box

By setting the baseline, a success notification appears. To unset the baseline deselect the check box.

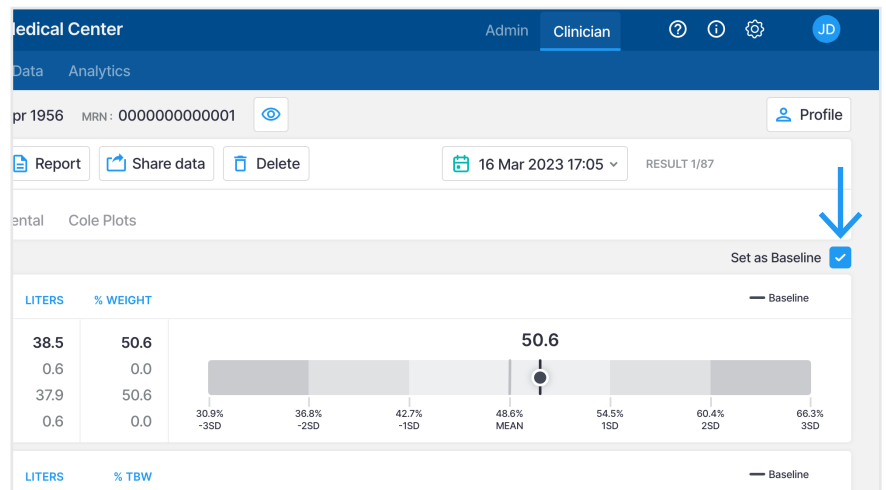


Setting the Baseline for Segmental

1

Select Measurement Date

1. Sign into MySOZO or SOZOapp as a clinician
2. Find and select a patient
3. From the measurement list, select the measurement to be used as a baseline


! "Measure" is only available through tablet

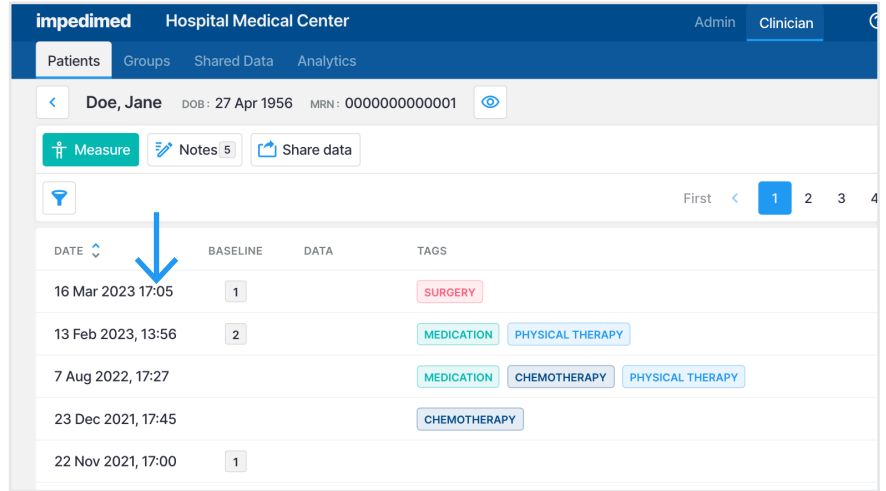

2

Select Segmental

1. From the results screen, select the Segmental tab

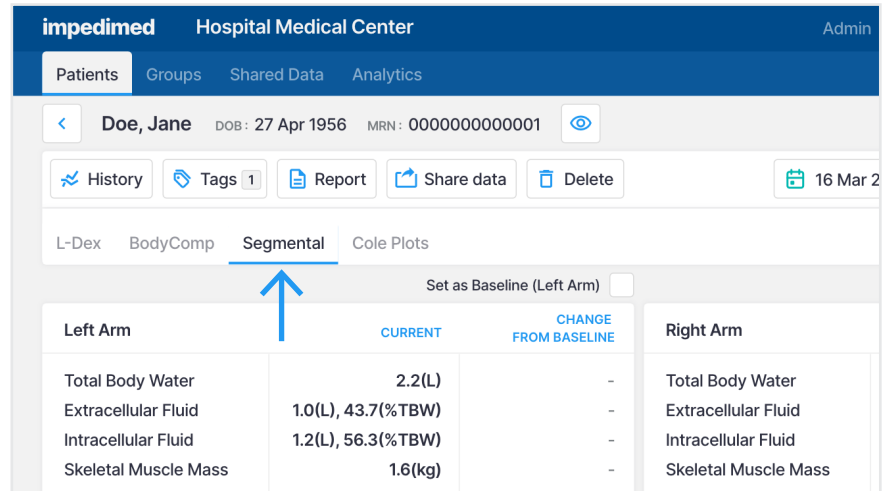


3

Set Baseline

1. Select the "Set As Baseline" check box

By setting the baseline, a success notification appears. To unset the baseline deselect the check box.

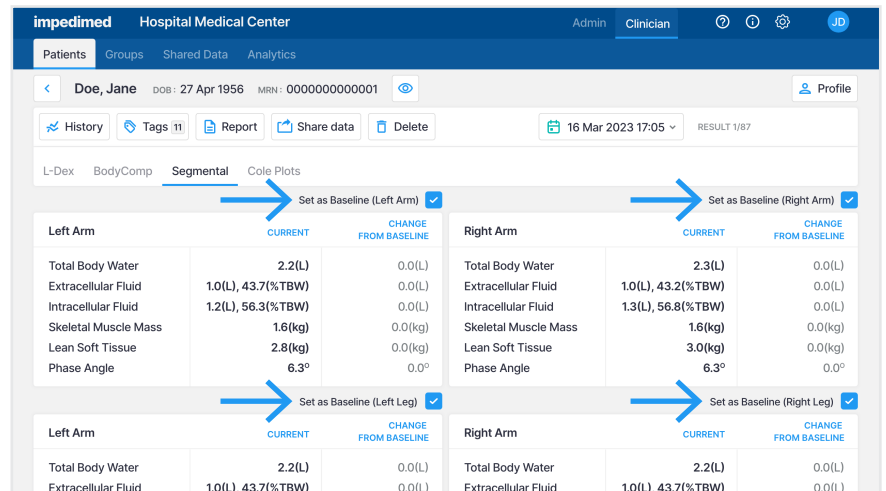

Please note: For each limb you have to set a separate baseline.