

## MYSOZO & SOZOAPP CLINICIAN


# Tags

1

## View Tags

1. Sign into MySOZO or SOZOapp as a clinician
2. Select a patient and navigate to a measurement
3. Select "Tags"

**Please note:** A tag must be first defined by an Administrator before it can be assigned to a measurement.

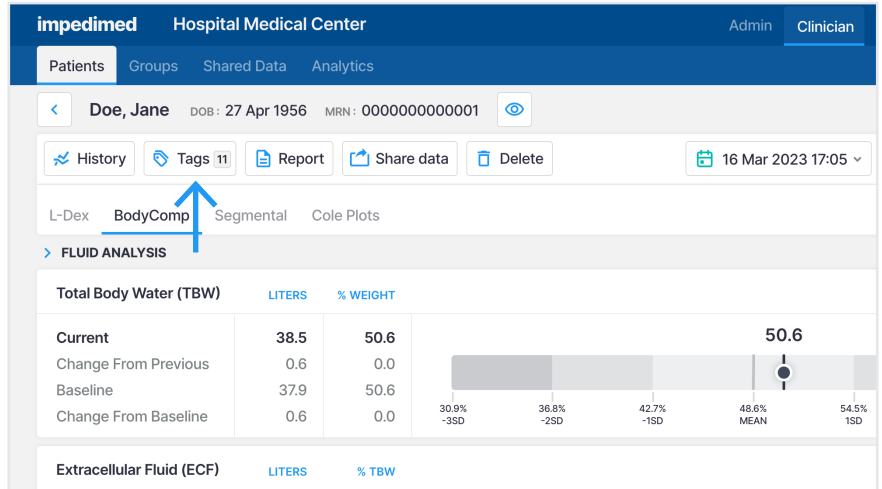


2

## Add New Tag

1. Select a tag from the drop down menu
2. Add tag description (Optional)
3. Select save icon

**Please note:** A tag can be assigned to a measurement just once. Those that are already assigned will be disabled.

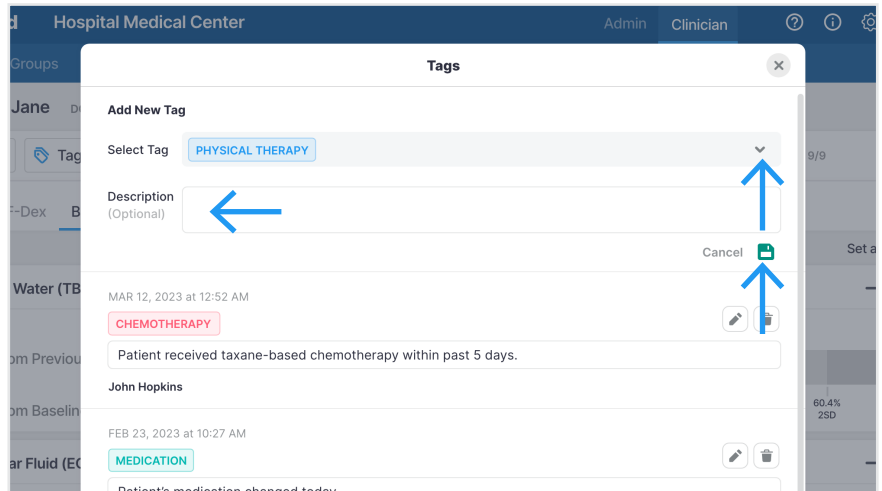

3

## Edit or Delete a Tag

### EDIT

1. Select the edit icon
2. Update the description
3. Select the save icon

### DELETE

1. Select the delete icon
2. Confirmation pop-up will appear. Select "Delete" to confirm deletion.

